

REUSABLE HYGIENIC FACE MASK

3 PLY

cosidos
treses

RULE:
UNE 0065:2020

WARNING: the hygienic mask should not be considered a sanitary product (PS) in order of directive 93/42 CE or regulation EU/2017/745, and not an individual protective equipment (PPE) in order of the regulation EU/2016/425

LICENSE NUMBER: 7785PS

SPECIFICATION:

The hygienic mask is destined for people without symptoms who are not sensitive to using surgical masks or filtering masks of protection against particles, according to measures established by the Ministry of health in the technical document "Prevention and control of infection in the management of patients with covid 19 ". It is also valid for children from 3 years old who have the conditions described above.

DESCRIPTION:

The hygienic masks cover the nose, mouth and chin. They are made of a suitable reusable filter material, which allows breathing and does not present known risks of irritation or adverse health effects in contact with the skin.

Assure a close fit with the face. Made to be reusable as long as the cleaning instructions are followed.

SIZES:

This product has been developed in four sizes: one for adults and three for children (from 3 to 5, from 6 to 9 and from 10 to 12).

CLEANING AND MAINTENANCE:

10 cycles of hand wash, in 60°C (UNE...) and air dry (UNE...).

It should be washed by hand, diluting 30 ml of bleach in 970 ml of water, hot at 60°C, immerse the mask leaving it in this solution for about 20 minutes, and let it dry in the air, hanging in a clean place.

Contact between a dirty mask and clean clothing should be avoided. The user in charge of the wash, must protect himself when handling dirty masks. It is recommended to let the mask dry completely for 2 hours after washing.

MASK TO PROTECT FROM RISK OF CONTAGION BY COVID-19



PRODUCT MADE IN SPAIN

It should not be dried or sanitized with a microwave. Environments in which the mask can become recontaminated during drying and storage should be avoided

A visual inspection (with gloves or washed hands) should be performed after each wash cycle. If damage is detected the mask should be thrown. Store in a clean and dry place and at a temperature between -20 and + 30 with a maximum relative humidity of 80. Do not modify or repair the mask.



All data and visual content that appears in this document is the exclusive property of treses cosidos and its copying, reproduction, alteration, diffusion, adaptation, integral or partial modification of it is prohibited. This document is private between treses cosidos and the receiver, so transfer to anyone is prohibited.

REUSABLE HYGIENIC FACE MASK



NORMA:
UNE 0065:2020

WARNING: the hygienic mask should not be considered a sanitary product (PS) in order of directive 93/42 CE or regulation EU/2017/745, and not an individual protective equipment (PPE) in order of the regulation EU/2016/425

LICENSE NUMBER: 7785PS

USE

Masks must be used correctly according to the following guidelines to be effective. For this reason, children should be supervised by adults for the placement, use and remove of this product.

It must be disinfected before the first use, following the washing instructions previously described.

It is recommended to place the mask without the presence of hair in contact with the user's skin and, for some people, shaved skin, and to respect the following steps:

- 1- Wash your hands with soap and water for at least 40 seconds or rub them with a hydroalcoholic solution before handling the mask.
- 2- Identify the upper part of the mask.



- 3- Position the mask on the face, placing the nose clip at the height of the nose.
- 4- Hold the mask from the outside and hold the headgear on both sides of the ears, without crossing them.
- 5- Lower the lower part of the mask to the chin.
6. Check that the mask covers the chin.
- 7- Pinch the nose clip with both hands to adjust it, to the nose.
- 8- Check that the mask is correctly placed. For this it is necessary to verify the seal and the absence of respiratory discomfort.



FEATURES

Main fabric,
double layer.

100% white polypropylene
(60 g/m² + 60 g/m²)

Inner fabric,
SPUNLAC
non-woven fabric

80% Polyester/20%viscose;
44 g/m²; what constitutes the middle
part of the mask

Packaging: bags of 25 masks

REMOVAL OF THE MASK

To avoid contamination when removing a mask, the following steps should be followed:

- 1 Remove protective gloves.
2. Wash your hands with soap and water or rub them with a hydroalcoholic solution.
3. Remove the mask without touching the front.
4. If the mask is to be reused, see the washing and maintenance section and the disposal section will be permanently discarded.
5. Wash your hands with soap and water or rub them with a hydro-alcoholic solution.

It is recommended not to use the mask for more than 4 hours for comfort and hygiene. The mask should be washed every time it gets dirty, wet, or cannot fit perfectly. If it is to be stored temporarily, contact with any surface, should be avoided. It should also not be placed in a holding position on the forehead or under the chin during and after use. once the maximum recommended number of washes is reached, the mask should be removed. This must be done in a container provided with a plastic bag (preferably with a lid and no manual control). It is recommended to use a double bag to keep the contents in the event of a tear in one of the bags. Used masks can also be disposed of in bio-waste containers.

